

No. 3112 FILTER REPLACEMENT INSTRUCTIONS

For the SurgiSonic® 1211X surgical instrument cleaning machine.

REPLACE ALL SIX FILTERS AND FLUID RETURN TUBES AT THE SAME TIME



NOTE: Wear appropriate apparel when handling contaminated items.

Every filter has a short and a long piece of plastic tubing permanently attached to it. The short piece of the plastic filter tubing is attached to the metal tubing on top of the pump housing. The long piece of the plastic filter tubing is attached to the suction chamber metal tubing.

- 1a) **To remove a used filter:** With machine off, gently twist and pry the short piece of plastic tubing off one of the metal suction tubes located on top of the pump housing.
- 1b) Next, pry the long piece of the filter plastic tubing off of the metal tube on top of the suction chamber. **Discard filter properly.**
- 2a) **To attach a new filter:** Dip the short end of the new filter plastic tube into water to lubricate and then twist it onto the metal suction tube on top of pump housing.
- 2b) Dip the long end of the new in-line plastic filter tubing into water to lubricate it and then twist it onto the short metal tube located on top of the suction chamber.