



SurgiSonic® Robotic Adaptor Kit

For cleaning the distal tool-end areas of robotic instruments.

NO. 3320 SurgiScrub® Scrubbing Chamber Kit contains:
3 No. 3315 SurgiScrub® Chambers for cleaning the distal tool end of robotic instruments.

1 No. 3316 SurgiScrub® Chamber cleans a robotic stapler.
24 No. 3200 Male to Male luer slip connectors.

The No. 3320 SurgiScrub® Scrubbing Chamber Kit is required to clean da Vinci® instruments.

NOTE: Because a tight seal separates the jaw/pulley end from the shaft/control box end, a patented dual hook-up method, using *two different types of suction chambers*, was created for independently cleaning both the distal and the proximal segmented areas of a da Vinci®* robotic instrument at the same time.

LEFT SIDE FACING MACHINE



Left side: Remove the *large* No. 3312 SurgiSuction® suction chambers, used for cleaning non-robotic tubular instruments, from the filter tubing on the left side. Next, attach three narrow diameter No.3315 multi-holed, **SurgiScrub®** scrubbing/suction chambers, for cleaning the tool end of robotic instruments, to the left filter tubing. The *large* No.3316 **SurgiScrub®** multi-holed scrubbing/suction chamber cleans a robotic stapler.



RIGHT SIDE




FACING MACHINE



Right side: Insert one end of a male to male luer slip connector into the number 2 output hole of a da Vinci® robotic instrument control box and insert the other end of the same luer slip connector into the large No.3312 SurgiSuction® suction chamber capped with a No. 3212 medium hole nipple. This applies maximum suction to the control box and shaft of the instrument. **NOTE:** Every attached tool end suction chamber and every attached control box end suction chamber is independently connected, via plastic tubing, to their own individually dedicated in-line filter, suction pump, and fluid return tube.

FULLY CONNECTED ROBOTIC TO BOTH SIDES OF MACHINE

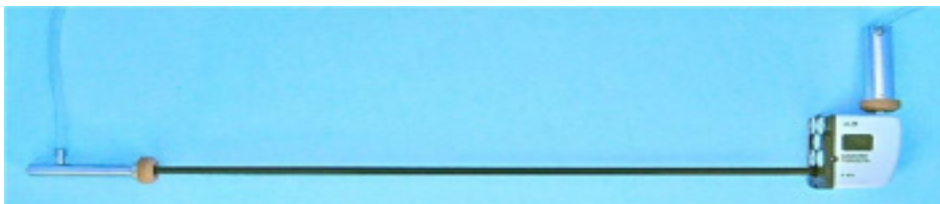

Fully connected da Vinci® robotic instrument ready for cleaning. Cleans three da Vinci® robotic instruments at a time.

*da Vinci® is a registered trademark of Intuitive Surgical, Inc.

SurgiSonic®, SurgiScrub®, and SurgiSuction® are registered trademarks of TBJ Incorporated.

TBJ Incorporated

1671 Orchard Drive, Chambersburg, PA 17201 Ph: 1-717-261-9700 Fx:1-717-261-1730

[e-mail: sales@surgiclean.com](mailto:sales@surgiclean.com)